

Verify™ Biological EO Test Pack

Biological EO Test Packs

for use in monitoring healthcare ethylene oxide (EO) gas sterilization processes



STERIS Verify Biological EO Test Packs are for use in monitoring healthcare ethylene oxide (EO) gas sterilization processes.

Each test pack is pre-assembled for convenience and ease of use, and utilizes the Verify Self-Contained Biological Indicator. Simply place the test pack in the center of the load in the sterilizer and run the standard cycle. Verify Biological EO Test Packs exhibit the same biological resistance pattern as properly assembled AAMI test packs, but with fewer steps and less chance of error.

EO Chemical Indicators in each test pack, as well as on each vial, provide immediate proof of processing. After incubation, a vivid color change in the growth medium from deep blue to bright yellow, or evidence of turbidity, gives unmistakable evidence of microbial growth.

With Verify Biological Indicators, results can be read with just 48 hours incubation after EO processing.

Incubators specially designed for use with Verify Biological Indicators are available in dual temperature (35-39°C for EO, 55-59°C for steam) and single temperature (35-39°C for EO only) models.

APPLICATIONS

- Ethylene oxide sterilization monitoring

STERIS®





INCUBATORS

Product #	Description
S3080	Verify 28 Vial Steam/EO Dual
S3082	Verify 28 Vial Steam
S3271	Verify 6 Vial Steam

STERIS Sterilization Process Monitoring...An Integrated Approach

The importance of efficiently monitoring the effectiveness of sterilization processes in the healthcare setting is universally recognized. STERIS has taken a systematic approach to this need and is committed to offering healthcare professionals an integrated family of fully compatible sterilization process monitoring products.

Verify Biological/Chemical Indicators

- Completely self-contained biological indicator for both steam and EO gas sterilization. Dual spore species (*Geobacillus stearothermophilus* and *Bacillus atrophaeus*) in a single vial, along with a specially formulated soybean casein digest growth medium.
- Steam and EO chemical indicators on each vial provide immediate proof of processing. A vivid color change in the growth medium from deep blue to bright yellow or evidence of turbidity gives unmistakable evidence of microbial growth.
- Read results with just 24 hours incubation after steam processing, or 48 hours incubation after EO processing.
- Available in packages containing 50 (S3061) or 100 (S3060) tests.

Verify Disposable Biological Steam Test Packs

- Convenient pre-assembled steam sterilization challenge test pack featuring the Verify Self-Contained Biological Indicator.
- Exhibits the same biological resistance pattern as properly assembled AAMI test packs, but with fewer steps and less chance of error.
- Read results with just 24 hours incubation after steam processing.
- Available in packages of 20 tests containing 5 controls (S3065) or 20 controls (S3069).

Verify Disposable Biological EO Test Packs

- Convenient pre-assembled EO sterilization challenge test pack featuring the Verify Self-Contained Biological Indicator.
- Exhibits the same biological resistance pattern as properly assembled AAMI test packs, but with fewer steps and less chance of error.
- Read results with just 48 hours incubation after EO processing.
- Available in packages of 20 test containing 5 controls (S3070) or 20 controls (S3071).

Verify Incubators

- Specially designed for use with Verify Biological Indicators. Available in dual temperature (35-39°C for EO, 55-59°C for steam) and single temperature (55-59°C for steam only or 35-39°C for EO only) models.



STERIS offers a complete line of Low Temperature Sterilization Systems including the Amsco® Eagle® 3017 EO Sterilization System.

STERIS Corporation is a leading provider of infection prevention, contamination prevention, microbial reduction, and surgical support systems, products, services, and technologies to healthcare, scientific, research, industrial, and government customers throughout the world.

STERIS Offices Worldwide

Canada	800 661 3937
Germany	49 2233 69990
Italy	39 02 2130341
Japan	81 78 321 2271
Korea	82 2 517 1517
Latin America	305 442 8202
Singapore	65 68 41 7677
Spain	34 916 585 920
United Kingdom	44 1256 840400

Publication ID #M1240EN.2003-09, Rev. E
GPSI Printed 09/2003, 7500
©2001-2003 by STERIS Corporation.
All rights reserved.

Technologies to Prevent Infection and Contamination™



STERIS Corporation
5960 Heisley Road
Mentor, OH 44060-1834 ■ USA
440-354-2600 ■ 800-548-4873
www.steris.com